boso

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Purpose

Non-invasive blood pressure meter to measure systolic and diastolic blood pressure levels in humans.

Calibration checks

Calibration checks must be performed in accordance with the national regulations in force.

EN

N.B.

Protect the device against severe shocks (do not bump into it or drop it), dirt and liquids. Make sure that the cuff is not damaged by sharp objects such as needles, scissors etc.

Do not inflate above 300 mmHg.

The reading time should not exceed 2 minutes. A gap of at least 2 minutes must be left between 2 readings.

Introduction

Two values need to be measured to determine blood pressure. **The systolic (higher) blood pressure.** This is produced when the heart contracts and pushes blood into the blood vessels. **The diastolic (lower) blood pressure.** This is produced when the heart muscle is relaxed and is refilling with blood.

Blood pressure readings are expressed in mmHg (mm mercury column). The following thresholds have been set by the World Health Organization (WHO). Ideally, these values should be based on the average of several different readings.

	systolic / mmHg	diastolic / mmHg
clearly elevated	140 or above	90 or above
high end of normal	130-139	85-89
normal	120-129	80-84
optimum	up to 119	up to 79

The reading process:

- **General principles:** The reading should be taken with the patient seated and relaxed. In individuals who have higher blood pressure in the left arm, readings should be taken on the left arm. However, in individuals who have higher blood pressure in the right arm, readings should be taken on the right arm.
- Place the bare arm on the table or armrest and bend it slightly.
- Fitting the cuff:
Open the cuff to form a ring shape and slip it over the left upper arm until the lower edge of the cuff reaches a point about 2-3 cm above the elbow. The cuff must be positioned so that the marking lies on the artery of the upper arm.
- Hold the manometer with your right hand around the pressure bulb, placing your thumb and forefinger on the valve screw. (for devices with a push-button valve: with your thumb on the push-button).
- Close the valve (by turning the screw to the right, or by pumping hard in the case of push-button valves) and inflate the cuff, observing the manometer all the time. Inflate until the manometer indicator is approximately 30 mmHg above normal systolic blood pressure.
- Open the valve with your thumb and forefinger (turn the screw to the left, or in the case of push-button valves: open the valve slightly by pressing your thumb lightly

on the push-button) and watch the manometer pressure reading fall slowly.

- Pressure should be reduced by 2 to 3 mmHg a second between systole and diastole, i.e. it should take about 20 to 25 seconds for the indicator to fall on the scale from 150 to 100.
- You hear the sounds described below through your stethoscope (the stethoscope built into the cuff or a separate stethoscope):
- **Read off the systolic (higher) blood pressure value on the manometer scale when you hear the first clear arterial tone (pounding sound) in the stethoscope.**
- **Read off the diastolic (lower) blood pressure value when you hear the last audible arterial tone.**
- When you have read off the blood pressure values, open the valve screw fully (turn the screw to the left or press the push-button as far as it will go) so that you can deflate the cuff quickly and completely.

Cleaning and disinfecting the device and the cuff:

Please use only a soft, dry cloth to clean the device. We recommend using microzid sensitive liquid disinfectant (Schülke & Mayr) to disinfect the device by wiping it, leaving the disinfectant on for at least 5 minutes. We recommend a disinfectant spray to disinfect the velcro on the cuff.

Operating conditions:

Ambient temperature 10°C to 40°C -
relative humidity 15 to 85%

Storage conditions:

Ambient temperature -30°C to 70°C -
relative humidity 15 to 85%

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BOSCH + SOHN GmbH u. Co. KG